

PRESTIGE DUPLIX

PRESTIGE DUPLIX Polyvinylsiloxane at low viscosity, specially developed for models duplicating in the Technical Laboratory.



TECHNICAL DATA

- Very high dimensional stability
- Fluidity and precision
- Mixing and working also without machinery
- Compatible with gypsum, investments and resins.
- Low final hardness (22 shore A)
- Pink colour
- Dimensional stability: 0,005%

MIXING

Using only the appropriate measuring scoops, measure out the two components from the jars. The base to catalyst mixing ratio is 1:1. Combine equal proportions of the base and catalyst until a uniformly coloured mass is formed.

ATTENTION

- The setting time is 5 minutes and 30 seconds.
- Close each jar immediately after use.
- Do not use the measuring scoops or jar lids interchangeably.

HANDLING TIME *

- Mixing: 45 seconds.
- Total working time: 5/6 minutes
- Setting time: 30min.

* The times reported begin with the mixing phase of the product at a temperature of 23° C. (73° F.). Higher temperatures will quicken the process, whereas lower temperatures will slow it.

STORAGE

Keep it securely closed, and store in a cool, dry place at a temperature between 4°C e 28°C (39°F - 82°F). Keep away from heat sources, sparks or flames.

The expiration date printed on the packaging refers to the unopened product still sealed by the production.

The serial number and expiration date are printed on each individual product container. When contacting the distributor, it is important to have this information on hand in order to properly and promptly identify a given product batch.

IMPORTANT: Do not use after the expiration date.

CAUTION

- Avoid eye and skin contact with the catalyst.
- In case of contact with the skin, carefully wash the area with soap and water.
- In case of contact with the eyes, flush thoroughly with water and consult a physician.
- The elastomeric materials are chemically resistant; therefore any contact with clothing, including lab-coats, should be avoided.

PACKAGING

Product Code/ Description 057001 - 1 kg base + 1 kg catalyst

VANNINI DENTAL INDUSTRY S.r.l.

Via di Campigliano 55/A 50012 GRASSINA (FI) – ITALY Phone +39 055 644698 Fax +39 055 644697
E-mail : info@vanninidental.com www.vanninidental.com