

PRESTIGE UNIVERSAL ADHESIVE



Universal adhesive specifically indicated to guarantee the adequate adhesion between the impression materials PROTESIL - PRESTIGE with the various types of impression trays.
Only for professional dental use.

INSTRUCTIONS FOR USE

The surface of the tray must be perfectly clean and dry.
To coat the internal surface of the tray with a thin layer of PRESTIGE UNIVERSAL ADHESIVE.
Drying for about 2 minutes to obtain a perfect adhesiveness of the impression material.
Apply the silicone.

WARNINGS

Prestige Universal adhesive must never come in contact with eyes and skin. In case of contact, wash off with plenty of running water.
Avoid inhalation of vapours and keep away from heat sources, sparks and open flame.
Close carefully the product after use.
The product is chemically resistant; therefore any contact with plastic surfaces or clothing should be avoided.
Store in a cool, dry place at a maximum temperature of 25°C/80°F.
Do not use after the expiration date.

PRECAUTIONS

R11: Easy inflammable
R20/21: Injurious for inhalation and contact with the skin
R38: Irritating for the skin
S2: Keep out of the reach of the children
S9: Keep in ventilated place
S13: Keep away from food, drinks and animal foodstuff
S16: Keep away from flames and sparks - Not smoking
S46: In case of ingestion look instantly for medical advice and show the container or label
S36-37 Wear protective clothing and gloves

PACKAGING

Cod. Product/ description 058005 Bottle of 10 ml.

VANNINI DENTAL INDUSTRY S.r.l.

Via di Campigliano 55/A 50012 GRASSINA (FI) – ITALY Phone +39 055 644698 Fax +39 055 644697
E-mail : info@vanninidental.com www.vanninidental.com